- GB** **Shower board - User manual**
- D** **Badewannenbrett - Gebrauchsanweisung**
- DK** **Badebrædt - Brugervejledning**
- I** **Tavola da vasca - Manuale d'uso**

*Item no.*  
*Art. Nr.*           **302030**  
*Varenr.*  
*Codice*

**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
www.mobilex.dk

**GB Shower board 302030**

**D Badewannenbrett 302030**

**Description & intended use**

The shower board is a medical device which intended use is to give stable & easy access and seating support when disabled persons use a bathtub. It is capable of being adjusted to suit the majority of bathtub widths.

**Security issues**

Only use the product for the purpose it is intended for. Don't climb on the product. Use the product only in good technical condition. Do not overload the product. Use the product only on flat, smooth bath walls. Always check stability of product before use. Make sure the product is adjusted to the correct width before use. WARNING! Be careful not to trap your fingers in one of the board's holes or slots.

**Assembly and use**

The handle must be fitted into the 2 locating holes at one end of the board. The screws are fixed to the handle from the underside of the board. Locate & secure the individual posts into the required positions at 4 long slots on the board to fit your bathtub width. The posts are fixed by the screws from underside of the board. Please ensure there is no sideways movement of the seat & that the post surface is in contact with the bath walls from inside. Both ends of the seat must be fully supported by the bath sides.

**Cleaning & maintenance**

Clean the device regularly & keep it clean. Use soft cloth, warm water & neutral household washing agents. Do not use abrasives & bleach based cleaners.

Technical data	
Outer dimensions	71 x 32 x 19 cm
For bathtubs with	inner width 41 - 64 cm
Weight	2,7 kg
Max. load	190 kg

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is designed and made in accordance with the technical documentation which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment**

## Guarentee terms

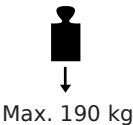
Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Lars Nygaard  
Managing Director  
Mobilex A/S

Skanderborg, 29th August 2017

signature

 <p>MOBILEX A/S Grønlandsvej 5 DK - 8660 Skanderborg Tel: +45 87 93 22 20 www.mobilex.dk</p>	<p><b>Shower board</b> <b>Item no. 302030</b></p>  <p>Max. 190 kg</p>
<p><b>CE</b> Produced <input type="text"/></p>	<p>Serial no. <input type="text"/></p>

Example of the product label

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The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark, declares that the: