- GB** **Round shower stool**
- D** **Runder Duschhocker**
- DK** **Rund badetaburet**
- I** **Sgabello tondo da doccia**

Item no. **302063 &**
Art.Nr. **302064**
Varenr.
Codice

User manual

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

GB Round shower stool

D Runder Duschhocker

Description & intended use

The round shower stool is a medical device which intended use is to give stable & easy seating support when disabled persons use the shower. It has an aluminium frame with height-adjustable legs and a round seat made of white plastic.

Important safety information

Only use the device for the purpose it is intended for. Don't stand on the device. Make sure that the legs are set to the same height. Use device only in good technical condition. Do not overload the device. Use device only on flat smooth surface with all 4 legs stably placed on the surface. Always check stability of device before use.

Assembly and usage description

The device 302064 is delivered assembled, with the four legs set to the lowest position. Adjust the seat height of the stool to fit the size of the user by pressing the pin-buttons on the stool legs and adjusting the legs to the desired height. Make sure that all four legs are set to the same height and that the buttons on all 4 legs are safely locked.

Difference and assembling of item 302063

The shower stool with item no. 302063 is the same product as 302064, but it is delivered disassembled. The package come with 2 set of legs (A and B), 1 white round seat (D) and 5 screws with 5 washers (C). To assemble it you should fix the legs to the underside of the seat, like shown in the picture on page 4. Take the white seat and lay it upside down on a table. Position the legs by paying attention to positioning leg A under leg B. Insert 1 of the 5 washers on each screw, then put the screws to the 5 holes on the underside of the seat and fix the screws. Take care that all legs are fixed and stable, then turn the shower stool to the floor. Check and regulate the legs to the desired height like described above.

Cleaning & maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents and soft cloth. Do not use solvents or abrasives.

Technical data	
Dimension	Ø 33 cm
Seat height-levels	35,5 - 53,5 cm
Max. user weight	130 kg
Product weight	1,66 kg

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Mobilex round shower stool Item no. 302063 & 302064

is designed and made in accordance with the technical documentation which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment**

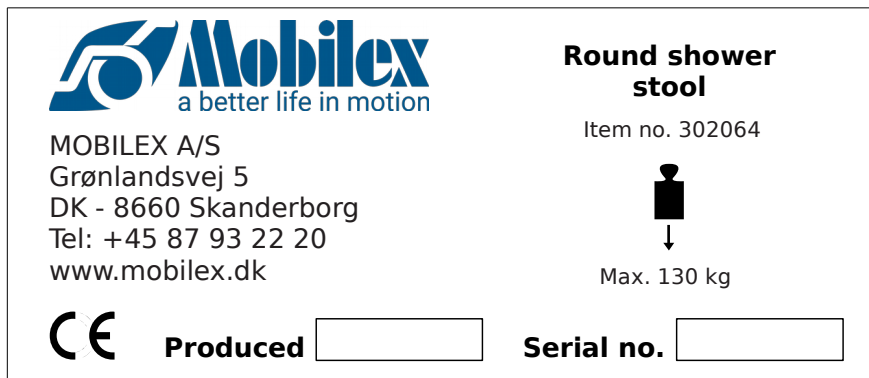
Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.


Lars Nygaard
Managing director
Mobilex A/S
Skanderborg, 29th August 2017

.....
signature

Example of the product label

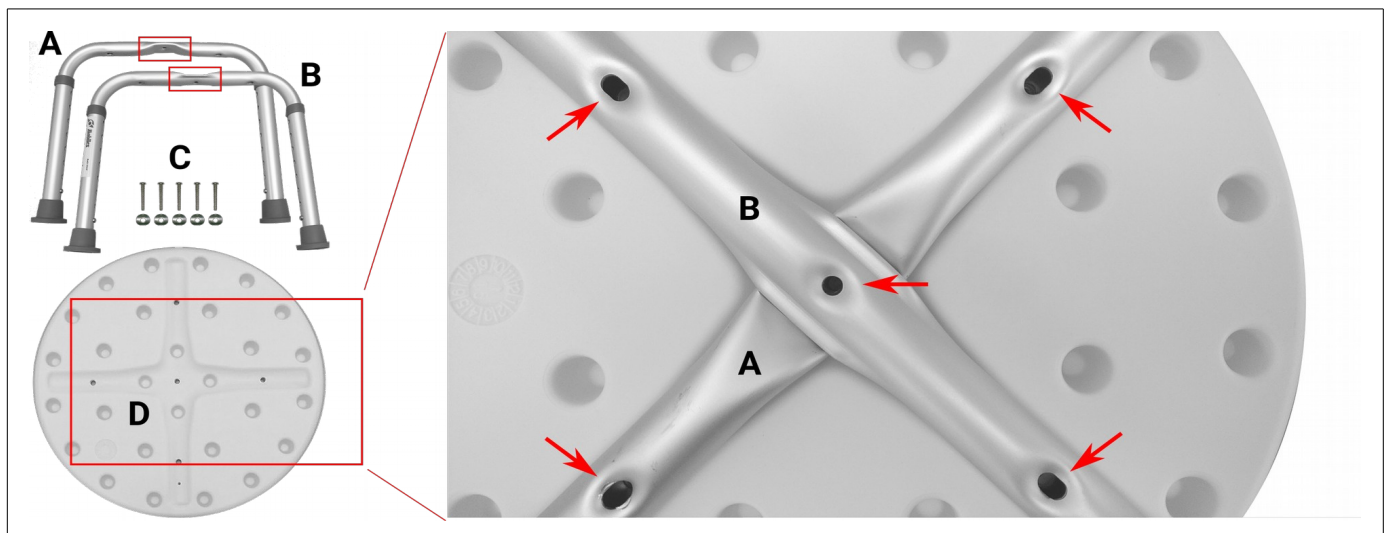Assembly of the 302063

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The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Danmark, declares that the: