- GB** **Triangular shower stool**
- D** **Dreieckiger Duschhocker**
- DK** **Trekantet badetaburet**
- I** **Sgabello triangolare da doccia**

Item no.  
Art.Nr. **302067**  
Varenr.  
Cod.

## User manual

**CE**

**MOBILEX A/S**  
Grønlandsvej 5  
DK-8660 Skanderborg  
www.mobilex.dk

**GB Triangular shower stool**

**D Dreieckiger Duschhocker**

**Description & intended use**

The triangular shower stool is a medical device which intended use is to give stable & easy seating support when disabled persons use the shower. It has an aluminium frame with height-adjustable legs and a triangular seat made of white plastic.

**Important safety information**

Only use the stool for the purpose it is intended for. Don't stand on the stool. It is important to sit at the centre of the seat! Make sure that the 3 legs are set to the same height. Use the stool only in a good technical condition. Do not overload. Use it only on a flat and smooth surface. Always check the stability of the stool before use.

**Assembly and usage description**

The stool is assembled by mounting the three legs on the underside of the seat. Adjust the seat height to the size of the user by pressing the pin-buttons on the stool legs and adjust the legs to the desired height. Make sure that all legs are set to the same height, and that the buttons on all legs are safely locked.

**Cleaning & maintenance**

Normal dirt on metal and plastic parts can be removed with standard cleaning agents and soft cloth. Do not use solvents or abrasives.

Technical data	
Dimension	50 x 45 cm
Seat height-levels	47 - 65 cm
Max. user weight	150 kg
Product weight	2,50 kg

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### **Mobilex triangular shower stool** Item no. 302067

is designed and made in accordance with the technical documentation which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment**

## Guarentee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Lars Nygaard  
Managing Director  
Mobilex A/S

Skanderborg, 29th August 2017

  
.....  
signature

	<b>Triangular shower stool</b>
MOBILEX A/S Grønlandsvej 5 DK - 8660 Skanderborg Tel: +45 87 93 22 20 www.mobilex.dk	Item no. 302067  ↓ Max. 150 kg
 <b>Produced</b> <input type="text"/>	<b>Serial no.</b> <input type="text"/>

Example of the product label

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The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark, declares that the: