AQUABAR støttegreb

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AQUABAR wall mounted grab bar

Instructions

Necessary tools: 1 power drill and 1 screw driver. Additional material necessary: 6 screws with rawl plugs of a shape and dimension suitable for the selected wall. **Attention:** the diameter of the threaded part of the screw must not exceed 4,4 mm and the head of the screw must have a diameter of at least 6 mm.

Instructions: Drill three holes in the selected wall with must correspond to the holes on the wall-mounting bracket. Insert the rawl plugs in the wall. Place the wallmounting bracket over the holes in the wall making sure the holes match. Insert the screws through the mounting brackets and into the rawl plugs.

Please note: remove the round plastic screw cover to view the holes on the wall-mounting bracket. Tighten the screws with the screw driver and check that the grab bar is securely fitted to the wall before pressing the round screw covers back into place. **Important:** should the wall-mounting bracket or the grab bar break or distort during installation substitute the article immediately.

Cleaning instructions: wash the grab bar without removing it from the wall. Wash with non-corrosive detergents.

Caution: make sure the selected wall is suitable to sustain the grab bar. Make also sure the grab bar is firmly fixed to the wall before use. Do not exceed max. weight indicated.

Technical details:

Material: ABS (wall-mounting brackets and screw covers) - PCV (bar)
Length: item no. 311230: 30 cm
item no. 311245: 45 cm
item no. 311260: 60 cm
Max. weight: 120 kg

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CE Mobilex A/S, Grønlandsvej 5