- Ⓚ **Schwellenrampe**
- Ⓒ **Doorstep ramp**
- Ⓓ **Dørtrinsrampe**
- Ⓘ **Rampa da soglia**

Art.Nr.
Item no.
Varenr.
Codice

DS2176

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
Danimarca
www.mobilex.dk

D Schwellenrampe DS2176

GB Doorstep ramp DS2176

Produkt Beschreibung

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Product description

Mobile and height-adjustable threshold ramp from aluminium with anti-skid surface. The supplied feet are adjustable in height.

Safety requirements

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Follow the information on the product label.
- Only use the product for the described purpose and only on even and flat surfaces.

Shipment

The ramp is supplied in a carton containing a plastic bag with 2 x 3 feet (3 pcs à 3,5 cm and 3 pcs. à 6 cm)

Assembly

- Remove the device from the box and choose the 3 feet that matches your need
- Screw the 3 feet to the ramp
- Make sure that the feet used are all of the same length
- Align the feet to the ground, so that all of them has ground contact
- Put the ramp in front of the doorstep you should pass over

Cleaning & mintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning (no solvents or abrasives).

Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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